CLINICAL CHEMISTRY STANDARDS

PRESENTACION	
REF	See description below Ver recuadro inferior
Only for <i>in vitro</i> diagnostic use Sólo para uso diagnóstico <i>in vitro</i>	

CLINICAL CHEMISTRY STANDARDS PATRONES DE QUIMICA CLINICA

INTENDED USE

These reagents are intended for the calibration of the quantitative determinations of clinical chemistry parameters.
The use of a standard to calculate results allow obtaining an accuracy independent of the system or instrument used.

REAGENT COMPOSITION

CAL Bilirubin Standard: Freeze-dried protein-based calibrator.
Hemoglobin Standard: Liquid animal-based calibrator.
Other standards: Liquid organic matrix based primary standards.

STABILITY

The standards are stable until the expiration date printed on the label when stored tightly closed and contamination during their use is avoided.

Storage and stability stated on the label.

Signs of deterioration

Presence of particles, turbidity and contamination.

PREPARATION

Liquid standards: Ready to use.

Freeze-dried standards: Reconstitution stated on the vial.

PROCEDURE

Follow the technical instructions of the corresponding kit.

NOTES

- Standards may cause a systematic error with some automated procedures when pumping devices are used.
The value or set point suggested by the manufacturer may not necessary be correct for the equipment and method used.
- Use clean disposable pipette tips for its dispensation.

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